



# TauroLock™ HEP 500

CE 0123

ANTIMICROBIAL CATHETER LOCK SOLUTION

# TauroLock™ HEP 500

CE 0123

Antibiotic  
Free

## Indication:

TauroLock™-Hep500 is approved to be instilled into catheter-based devices for hemodialysis in order to maintain patency of the vascular access device. Additionally, by creating a hostile environment to bacterial and fungal growth, the risk of catheter-related bloodstream infections (CRBSI) is reduced.

## Ingredients / Specification:

TauroLock™-Hep500 contains anticoagulant and antimicrobial substances. Active ingredients in TauroLock™-Hep500 are taurolidine, citrate (4%) and heparin (500 IU/mL). The product is sterile filter processed and supplied as a clear, sterile, non-pyrogenic solution. TauroLock™-Hep500 is to be used with a catheter-based vascular access device. It is to be instilled into the device lumen between treatments (dialysis).

## Instillation:

After treatment, the system is thoroughly flushed with saline (min. 10 mL). Then, TauroLock™-Hep500 is instilled into the access system by exactly respecting the filling volume. TauroLock™-Hep500 remains in the system until the next treatment. The solution must be aspirated before initiating the next treatment. Flush the device with 10 mL of saline.

## Contraindications:

TauroLock™-Hep500 is contraindicated for patients with a known allergy to taurolidine, citrate or heparin, and for patients currently taking medication with known adverse interaction to taurolidine, citrate or heparin. TauroLock™-Hep500 is also contraindicated for patients with heparin-induced thrombocytopenia or increased bleeding risk.

## Storage:

TauroLock™-Hep500 must be stored at 15 – 30 °C. Do not freeze. It must be neither transported nor stored at freezing conditions. Ampoules are for single-dose only, vials are multi-dose and must be used within 48 hours after the first puncture.

## Note:

This information does not replace the instruction for use.



article-no.	description	packaging
TP-02-5	TauroLock™-Hep500 catheter lock solution 5 ml ampoule (single-dose)	10 x 5 ml
TP-02-10	TauroLock™-Hep500 catheter lock solution 10 ml vial (multi-dose)	100 x 10 ml

Manufacturer:

**TauroPharm**  
GmbH

TauroPharm GmbH  
August-Bebel-Str. 51  
D-97297 Waldbüttelbrunn  
Tel.: +49 (0) 931 304 299-0

ISO 13485

[WWW.TAUROLOCK.COM](http://WWW.TAUROLOCK.COM)

85102/13/20